

PERFORMANCE OF THE SYRINGE



Obr. 1

There's a sealing at the down end of the dart



Obr. 2

Press there with both fingers and you'll make it transit



Obr. 3

While pressing the down sealing by applied venting pin, push upper piston of the syringe



Obr. 4

The dart is ready for filling



Obr. 5

Fill the dart with the active substance



Obr. 6

Put the sealing on the needle



Obr. 7

Screw the needle on the dart



Obr. 8

Attach syringe for pressuring by using coupling, push the lower sealing (like it was shown on pic. 2), saturate it with air and free the lower sealing



Obr. 9

If needed push more air and make sure that sealing is put on the needle correctly. The dart is ready for use, put it in the blowpipe. Aim at your target and by rapid blowout shoot the dart out off the blowpipe

